
HEARING AMPLIFIER

Instruction Manual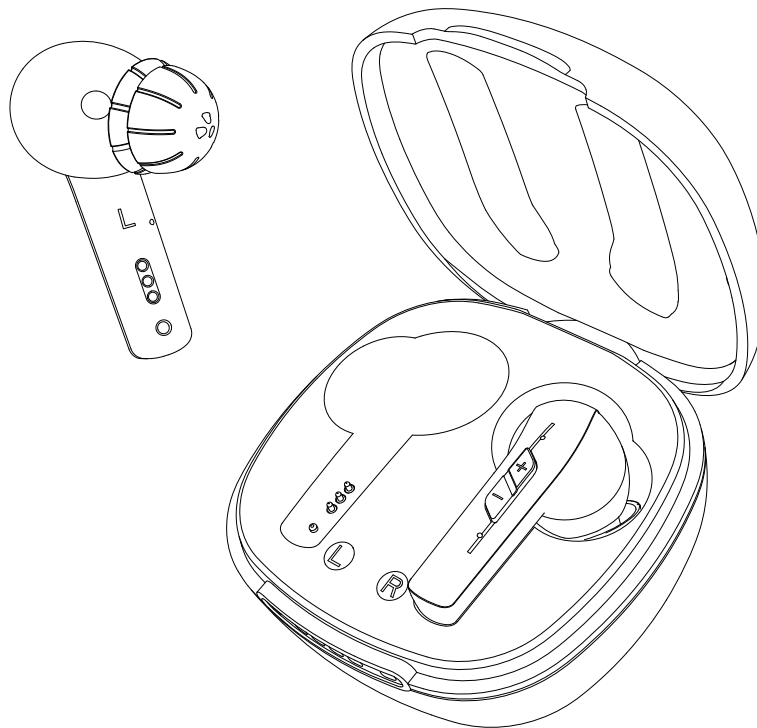

Applicable to JH-A32E model

Please read this user manual thoroughly and completely
before using the product.

Contents

I. IMPORTANT SAFETY INSTRUCTIONS.....	1
1. CONTRAINDICATION.....	1
2. WARNINGS.....	1
3. NOTICES.....	1
II. DO NOT USE IF ANY OF THE FOLLOWINGS APPLY TO YOU.....	2
III. MATTERS NEEDING ATTENTION, WARNINGS AND TIPS.....	2
IV. MAINTENANCE AND CLEAN.....	3
1. Daily maintenance.....	3
2. Care and maintenance.....	3
3. Cleaning.....	3
V. HEARING AMPLIFIER CARE.....	3
1. Hearing amplifier care.....	3
VI. NORMALIZED SYMBOLS.....	4
VII. MAINTENANCE, STORAGE, AND DISPOSAL.....	4
VIII. WARRANTY.....	5
IX. HOW TO USE YOUR DEVICE.....	5
1. [GET TO KNOW YOUR DEVICE].....	5
2. [ACCESSORIES].....	6
3. [USING THE HEARING AMPLIFIERS].....	6
4. [USING THE CHARGER].....	11
5. [HOW TO CARE FOR YOUR DEVICES].....	12
6. [FURTHER INFORMATION].....	13
7. [TECHNICAL SPECIFICATIONS].....	14
X. CAUTION.....	15
EN.....	1
FR.....	16
DE.....	32
IT.....	48
ES.....	64
RU.....	80
PT.....	96
JP.....	110

I. IMPORTANT SAFETY INSTRUCTIONS

Read all the information in this instruction manual and any other literature included in the box before using the device.

1. CONTRAINDICATION

The device should not be used in the following situations:

- Patients with chronic suppurative otitis media (CSOM), congenital ear atresia, or any type of ear discharge.
- Anyone under 36 months old.

2. WARNINGS

- Keep small parts (hearing instrument, eartips, battery or other detachable parts) that can be swallowed out of children's reach. Dropping, immersing in liquid, exposure to strong electromagnetic fields or excessive heat will damage the hearing amplifier. Remove devices prior to any type of X-ray, MRI, or CT scan. If parts become stuck in the ear canal, seek medical assistance—do not attempt to remove by yourself.

3. NOTICES

- Do not use at high volume for extended periods of time – ear injury can result.
- If you experience any pain or strong discomfort using the devices, remove and consult a doctor.
- Routine maintenance to keep the device clean is required. Ensure that eartips are free of debris and moisture at all times.
- Avoid shocks or dropping the product.
- Do not store or use the product in environments of high humidity or temperature (such as sauna room), to avoid damage.
- Batteries should be properly recycled and not disposed in the trash.
- Do not set volume levels too high. Listening with high-output devices for extended period of time at high volume levels can cause further hearing damage.
- Do not share your hearing amplifiers with others. Using a hearing amplifier configured for someone else can result in hearing damage or injury.
- Monitor for any unusual skin reaction. If you notice any allergic reaction (itchiness, swelling, or rash), stop wearing the hearing amplifiers and consult a medical specialist.
- Do not touch the hearing amplifiers to any type of magnet or magnetized object. Your hearing amplifiers are delicate electronic instruments and may be damaged by contact with strong magnetic fields.
- Do not attempt to open or repair the devices yourself. Damage can result and will void the warranty.
- Do not store your devices in a hot/humid locations. You should never leave your hearing amplifiers near heating appliances or in direct sunlight.
- Do not leave your hearing amplifiers in the car as inside temperatures can reach dangerous levels which could damage the devices.

If the Bluetooth function is available for your device, do NOT connect the hearing amplifier and headsets to the mobile phone via Bluetooth simultaneously - noise, distortion or any other unexpected problems may occur to your hearing amplifier.

- Please contact the MANUFACTURER:

• For assistance, if needed, in setting up, using or maintaining your devices or to report any unexpected operation or events.

Specific warnings / statements
Keep small parts away from children to avoid accidental ingestion.
Do not use the devices in explosive or oxygen-enriched atmospheres, as doing so may damage the devices.
Warning that it should only be used by intended person as using hearing amplifiers designed for someone else may result in hearing damage.

II. DO NOT USE IF ANY OF THE FOLLOWINGS APPLY TO YOU

1. Congenital atresia of external ear or deformity of any part of the ear canal.
2. Deafness is unstable, and the degree of hearing loss often fluctuates.
3. Persistent headaches, dizziness, earaches, tinnitus or other symptoms that are not suitable for adaptation.
4. Sudden onset of hearing loss within the past 3 months; rapid progressive deafness; unilateral hearing loss or other acute ear disease.
5. Effusion (secretions/discharge) in the ear and/or frequent purulence (pus).
6. Central deafness (central nervous system) or non-organic deafness.
7. Patients with acute otitis externa and tympanitis.
8. Patients with acute and chronic suppurative otitis media (in the period of purulent infection).
9. Patients who are allergic to the materials of this product.

III. MATTERS NEEDING ATTENTION, WARNINGS AND TIPS

1. Do not immerse the hearing amplifier in any liquid. Remove the hearing amplifier before swimming or bathing. After swimming or bathing, wait for the ear canal to dry before wearing the hearing amplifier.
2. The external surfaces of the hearing amplifier should be kept clean, and earwax in the eartips should be cleaned out daily.
3. If the device is rechargeable, please charge the devices regularly to avoid battery damage; The recommended charging interval of the devices is less than 28 days.

IV. MAINTENANCE AND CLEAN

1. Daily maintenance

- Keep your devices clean and dry by using soft cloth or tissue, to avoid potential damage from humidity or perspiration.
- Do NOT use the electronic dryer to store the rechargeable hearing amplifier.

2. Care and maintenance

- Keep your devices away from children and pets.
- Keep your hearing amplifier away from high-temperature and/or high-humidity atmosphere.
- Do not poke the microphone with any sharp objects to avoid damage.
- Do not attempt to disassemble or repair your devices by yourself. Doing so will void your warranty.

3. Cleaning

1) Cleaning the eartips

- ① Clean up earwax adhered to the eartips by using the thin side of enclosed tool.

*Remove eartips from the hearing amplifier before cleaning, and push the eartips rim membrane outward.

- ② Clean the eartips with alcohol and dry cloth and wait for the eartips to dry before attaching them to the hearing amplifier.

*Note:

- We recommend that changing the eartips every three months.
- If the eartips get stiff, brittle or discolored over time, please change them as soon as possible.

2) Cleaning Tool

Using soft cloth or tissue to wipe any moisture, perspiration from the surface of the cleaning tool.

V. HEARING AMPLIFIER CARE

1. Hearing amplifier care

- Keep your hearing amplifier clean. Extreme temperature (heat or cold), moisture, and foreign substance can result in poor performance.
- Use the cleaning tool to clean debris adhered to the buttons, microphones or other parts; inspect the receiver, eartips or other parts regularly.
- Do NOT clean your hearing amplifier with water, solvent or oils.

*NOTE: Do NOT fix your devices by yourself. Doing so will not only void the warranty but also cause further damage easily. Check the "Troubleshooting" first when failures occur to your devices. Contact your hearing professional for assistance if needed.

VI. NORMALIZED SYMBOLS

	Follow operating Instructions		
	Disposal in accordance with Directive 2012/19/EU (WEEE)		
	CE Labeling This product satisfies the requirements of the applicable European and national directives.		
	Date of manufacture		Manufacturer information
	Serial number		USE BY
	Warning/ Danger: Improper use might cause serious injuries.		
Storage/Transport 	Permissible storage and transport temperature and humidity.		
Operating 	Permissible operating temperature and humidity.		
	Keep Dry		Comply with Directive 2011/65/EU
	Paper Packaging Recycling		

VII. MAINTENANCE, STORAGE, AND DISPOSAL

Maintenance

- 1) We do not authorize any institution or individual to maintain and/or repair the product. If you suspect that the products may be in question, please contact us immediately.
- 2) The user must not attempt any repairs to the device or any of its accessories. Please contact us for any repair needs.
- 3) Opening of the equipment by unauthorized agencies is not allowed and will terminate any claim to warranty. Warning: No modification of this equipment is permitted.

Storage

- 1) Do not store or expose the devices to extreme hot/cold temperatures, moist/humid environments, or shock/vibration.
- 2) Keep the devices in following environmental ranges: -10°C ~55°C, 10%-90%RH, 700-1060hPa.

Disposal

Please dispose of the device in accordance with the directive 2012/19/EU – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please refer to your local authority responsible for waste disposal.

VIII. WARRANTY

This limited warranty covers defects in material and workmanship for a period of one (1) year from the date of purchase, which includes the hearing devices, within the warranty period.

Any damage caused by improper handling and care, exposure to elements or chemicals, or unauthorized service will not be covered by the warranty. All other damage claims excluded. A warranty claim must be submitted with the purchase receipt

Henan Eshow Electronic Commerce Co., Ltd.

Add: 7/F, Sanjiang Building, No.170 Nanyang Road, Huiji District, Zhengzhou, Henan, China


E-mail: support@retekess.com

Web: www.retekess.com

IX. HOW TO USE YOUR DEVICE


1. [GET TO KNOW YOUR DEVICE]

1.1 Hearing amplifiers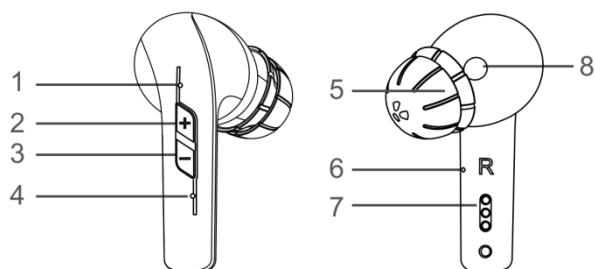

1. Microphone
2. Volume increase/Hearing program switch
3. Volume decrease
4. Microphone
5. Eartip
6. Microphone
7. Charging contact
8. In ear detection sensor

1.2 Charging case

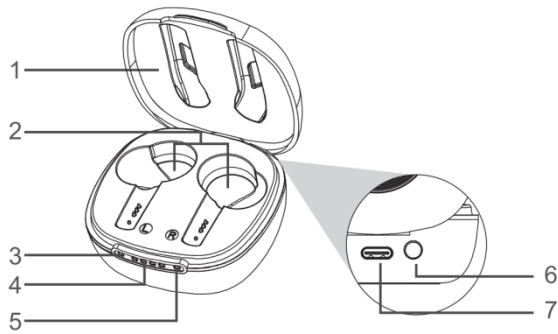1. Upper cover
2. Charging slots
3. Indicator light for hearing amplifier(L)
4. Indicator light for charging case
5. Indicator light for hearing amplifier(R)
6. Power switch of charging case
7. Charging port

1.3 Left & Right hearing amplifier markings

• There is a marking on the hearing amplifier, which helps to distinguish between left and right hearing amplifier.

L marking for left hearing amplifier.

R marking for right hearing amplifier.

2. [ACCESSORIES]

Items	Description
Eartip x 6	hold the hearing amplifier comfortably and securely in the ear canal.
USB Cable x 1	used to charge the charging case.
Cleaning Tool x 1	used to clean hearing amplifiers and charging case.
Manual x 1	used for users to read and understand how to use hearing amplifiers and other objects.

3. [USING THE HEARING AMPLIFIERS]


3.1 Fitting eartips

Please follow the following steps to apply eartips. The eartips may remain in the ear canal with incorrect replacement while taking the hearing amplifiers off the ear, which will cause hearing damage.


Step 1: Choose a suitable size of eartips.

Step 2: Gently pull the eartips rim membrane outward.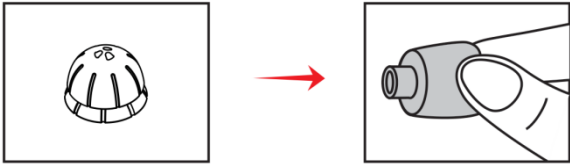

Step 3: Hold on the end of the hearing amplifier and push firmly to ensure that the eartips are fastened securely.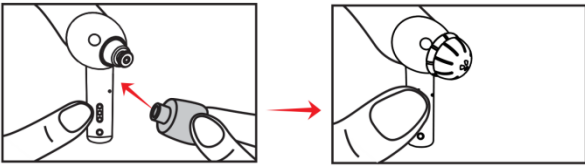

Notes: Please pull the eartips gently to check if they are installed firmly and ensure the eartips won't fall off easily.


3.2 ON/OFF

3.2.1 TURN ON

- The hearing amplifiers will turn on automatically when they are worn correctly. You will hear prompt tone according to the hearing program (refer to "Programs switching").

3.2.2 TURN OFF

- The hearing amplifiers will enter standby status automatically when they are removed from your ear.
- When placed back into the correct slot of the charging case, hearing amplifiers will turn off and start charging automatically.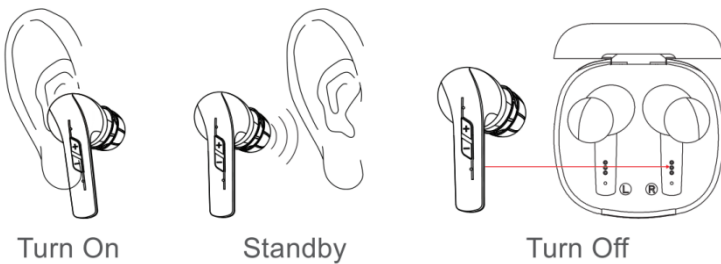


3.3 Wear/Remove hearing amplifiers


- Your hearing amplifiers are equipped with in-ear detection sensor.
- The hearing amplifiers will enter the standby status automatically when removed from your ear.
- The hearing amplifiers will turn on automatically when placed back in your ear.

3.3.1 Wearing hearing amplifiers

Step 1: Hold the hearing amplifier and place it in your right or left ear.

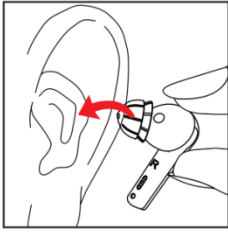
Step 2: Move the hearing amplifier slightly into your auricle for holding.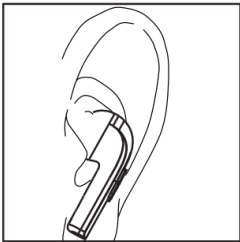

3.3.2 Removing hearing amplifiers

Step 1: Grasp the hearing amplifier and remove it from your auricle gently.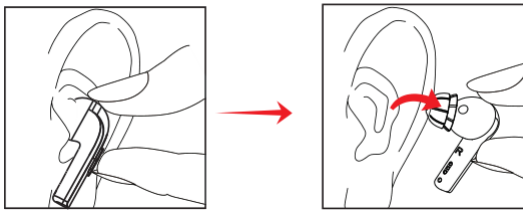


Notice:


- Hearing amplifiers will whistle when inserted incorrectly. Reinserting them into your ear canal correctly.
- If whistling remains after reinsertion, replace the eartips with bigger size and try it again.
- If the hearing amplifiers doesn't work properly, please check the "Troubleshooting". If none of the solutions works, contact our service team.

3.4 How to change the wax guard


- Wax guard helps to protect the hearing amplifiers from earwax.
- Wax guard is a disposable good that should be thrown away after used.
- The manufacturer do NOT supply additional wax guards with your hearing amplifiers. If you need it, please buy it from your healthcare professional or somewhere else.
- Please follow the following steps to change the wax guard.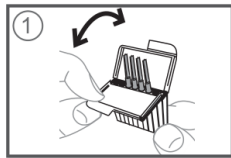


1. Open the wax guard case as illustrated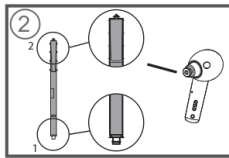




2. Take out the earwax baffle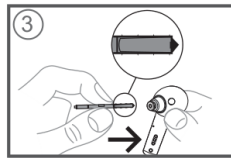



3. Hearing amplifier wax guard position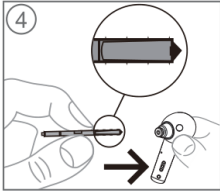


4. Insert the removal hook into the used wax guard. Keep the holder at a right angle. Pull the used wax guard straight out.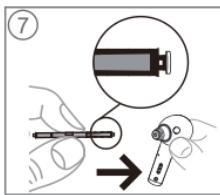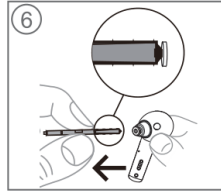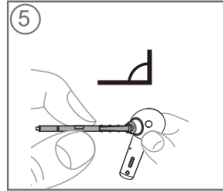

5. Insert the new wax guard into the opening of the sound outlet and press it gently, and pull the holder straight out.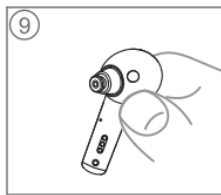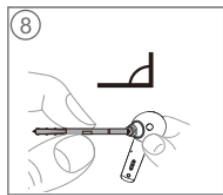

- Throw away the used wax guard.
- If the wax guard can not be locked on the hearing amplifiers, please replace the new wax guard.

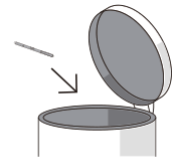3.5 Volume control

The hearing amplifiers have eleven (11) volume levels.

Short press the "+" button to increase the volume.

Short press the "-" button to decrease the volume.

The hearing amplifiers will give out the prompt tone "Beep-Beep-Beep" at maximum or minimum volume.

3.6 Programs switching

- There are four (4) preset programs in the hearing amplifiers.
- You could switch the hearing programs by long pressing the "+" button.
- Different programs have different beeps as below.

Hearing program	Prompt tone
Program 1	Beep
Program 2	Beep Beep
Program 3	Beep Beep Beep
Program 4	Beep Beep Beep Beep

- The hearing amplifiers are designed with the memory function: The last setting can be remembered when the hearing amplifiers turned off.

3.7 Bluetooth (2400-2484.5 MHz, Max 1 mW RF power)

Bluetooth® is a registered trademark owned by the Bluetooth SIG, Inc.

3.7.1 Bluetooth connection

- Be sure Bluetooth is available on your phone.

Step 1: Turn on your hearing amplifiers before connecting Bluetooth.

Step 2: Check the Bluetooth setting on your phone to "Pair New Device".

Step 3: Select device name "HearingSense 1" to pair (connect) the Bluetooth of your phone.

Connection status	Prompt tone
Connect successfully	Connected
Disconnect	Disconnected

- Accept any prompts given by your phone regarding the Bluetooth connection.

Notice:

Once the first pairing is done, the hearing amplifiers will reconnect to your phone automatically as long as Bluetooth of your phone is available.

Attention:

- Do NOT use the Bluetooth function of your hearing amplifiers for any medical purposes.

3.7.2 Bluetooth function description

(i) Listening to music

- You could increase the music volume by short pressing button "+" once.
- You could decrease the music volume by short pressing button "-" once.

(ii) Phone calls

- You could answer the incoming call by short pressing the button "-" once.
- You could reject the incoming phone or hang up the phone call by pressing the button "-" for about 1.5 seconds.

3.8 How to use app

Read all the information in this instruction before using.

3.8.1 HearingSense APP

Step 1: Tap to start "App Store" or "Google Play Store" on a compatible Apple or Android device.

Step 2: Search "HearingSense".

Step 3: Tap "GET" to download the "HearingSense" APP. Follow the system prompts to finish the installation.

Step 4: Tap to start the "HearingSense" APP.

Step 5: Follow the novice guide to finish the setup.

Step 6: Start to personalize and fine-tune your hearing amplifier.

Hearing Sense

- Further instruction is available on HearingSense APP ("More" → "Help").
- Requirement for running the app: iOS 14 or later; Android 5.0 or later.

*iPhone® is a trademark of Apple Inc., registered in the U.S. and other countries.

Android™ is a trademark of Google Inc.

iOS is a trademark of Cisco Inc.

3.9 Hearing testing

Welcome to use hearing amplifier and HearingSense APP. This app allows you to perform a complete hearing test with your compatible hearing amplifiers. The APP will automatically configure the hearing amplifiers based on the results of your hearing test.

3.9.1 Connect the hearing amplifiers

- 1) While wearing the hearing amplifiers have your Mobile phone ready and ensure that the Bluetooth function on the phone's system settings is connected to the hearing amplifiers.
- 2) Enter the "Bluetooth Devices" page from the homepage of the APP and connect to both the left and right hearing amplifiers.

3.9.2 Find a quiet environment in which to perform the test

- The APP will automatically detect the base noise level of the surrounding environment. If the ambient noise level is too high, the hearing test cannot be performed and the APP will not proceed to the next step. Please find a quiet environment to continue (Figure 1).
- We will test your hearing at six points: 250Hz, 500Hz, 1kHz, 2kHz, 4kHz, 6kHz, You can choose "Yes" or "No" according to the actual situation. When you make a wrong choice, you can withdraw it by "Undo" (Figure 2).

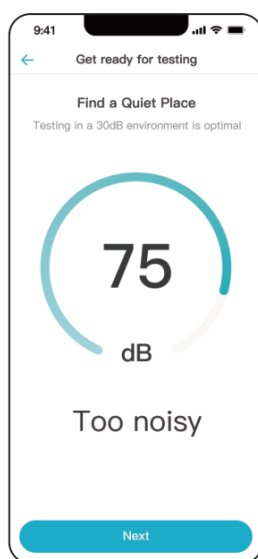
Figure 1

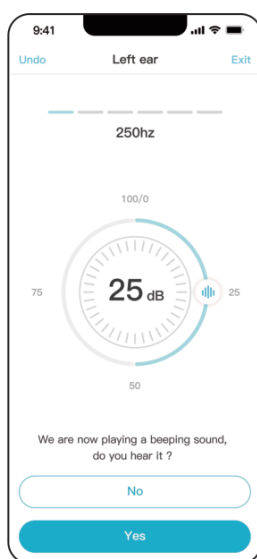
Figure 2

3.9.3 Save and set automatically

After completing the hearing test, the app will auto-fitting your hearing amplifiers according to your hearing test result.


4. [USING THE CHARGER]

4.1 Charging the hearing amplifiers

- The hearing amplifiers will turn off and start charging automatically when placed back into the correct slot of the charging case.
- The blue indicator lights indicate that the hearing amplifiers are charging and the white

indicator lights indicate that the hearing amplifiers are fully charged.

4.2 Charging the charging case

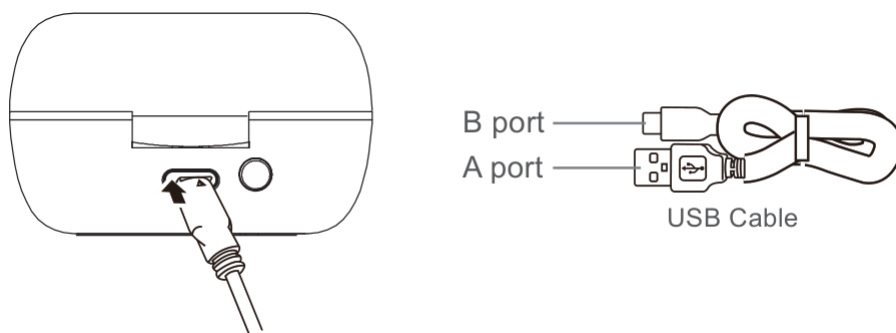Step 1 : Connect the B port of the USB cable to the charging port of the charging case.

Step 2 : Connect the A port of the USB cable to the power supply.

 Blue White Stop			
Device	Working	Charging state	Charging indicator of the charging case
Hearing amplifier	Hearing amplifier back into the charging case	Charging	Blue solid light
		Fully charged	White solid light (Automatically shut off after 5 mintues)
Charging Case	Hearing amplifier back into the charging case and the charging case unplugged from power	100%	
		75%~100%	
		50%~75%	
		25%~50%	
		0%~25%	Flashing
	The charging case will charge via the USB port on the case	100%	
		75%~100%	
		50%~75%	
		25%~50%	
		0%~25%	Flashing

Charging Indicator Changess of the Charging Case

*Note:

When the power level of the hearing amplifier is too low, the hearing amplifier will give out prompt tone.This indicates that the current hearing amplifier power is about to run out, please recharge it in time.

5. [HOW TO CARE FOR YOUR DEVICES]

Please follow the advice below for the best user experience and to prolong the life of your devices.

- Do not drop your hearing amplifiers or knock them against hard surfaces.
- Do not use sharp tools to dislodge ear wax.Sticking household items into your devices can seriously damage them.
- Keep your devices away from high-temperature and/or high-humidity atmosphere.

- Do not swim, shower or sauna while wearing the hearing amplifiers.
- Do not attempt to dry the hearing amplifiers or charging case in an oven, microwave or other heating equipment. This will cause them to melt and may cause burns to your skin.
- Always remove your hearing amplifiers when using hair care products. The hearing amplifiers can become clogged and cease to function properly.
- Store your devices in a clean and dry location
- Do not use the electronic dryer to store the rechargeable hearing amplifiers.
- Inspect the hearing amplifiers, eartips, and charging case regularly. Remove earwax and debris from your devices by using enclosed cleaning tools.
- Clean the charging contacts of the rechargeable hearing amplifiers with cleaning cloth regularly to ensure the normal charging.
- Eartips should be replaced every 3-6 months or when they become stiff, brittle, or discolored.

6. [FURTHER INFORMATION]

6.1 Troubleshooting

6.1.1 Hearing amplifiers fault self-checking table

Issue	Possible Causes	Potential solution
Feedback or "whistling"	Are the hearing amplifiers worn correctly in your ears?	Reinsert the hearing amplifiers in your ears.
	Is the volume very loud?	Reduce the volume.
	Are the eartips broken or clogged?	Replace the eartips
	Are you holding an object close to the hearing amplifiers?	Move your hand away to create more space between the hearing amplifiers and the objects.
	Is your ear full of wax?	Visit your physician.
No sound	Pressing time is too short to turn on hearing amplifiers.	Turn on your hearing amplifiers in the right way.
	Remaining battery capacity is used out.	Charge the hearing amplifiers.
	Microphone is clogged.	Clean debris or earwax from microphone.
Voice is weak or fuzzy	Low battery	Charge the hearing amplifiers.
	Volume is too low.	Increase the volume level.
	Microphone is clogged.	Clean debris or earwax from microphone.
Scan or search the Bluetooth of hearing amplifiers unsuccessfully	Misrecognize the Bluetooth broadcast name.	Bluetooth broadcast name: HearingSense 1.
	Bluetooth function of your mobile device is OFF.	Turn on the Bluetooth function of your mobile device.
Scan or search the Bluetooth of hearing	Hearing amplifiers is occupied by other mobile	Disconnect from other mobile device and try to connect again.

amplifiers unsuccessfully	device.	
Can not operate and adjust your hearing amplifiers via APP after connecting to your mobile device successfully.	Hearing amplifiers are not connected to the APP.	Connect your hearing amplifiers to the APP.
Music or phone function is not working after connecting successfully.	Hearing amplifiers do not get the permission of your mobile device about media or phone.	Set permission to your hearing amplifiers.
	Do-not-disturb mode of your mobile device is ON.	Turn off the Do-not-disturb mode of your mobile device.

6.2 Charging case fault self-checking table

Issue	Possible Causes	Potential solution
Indicator light of charging case is not light up or flashing.	Adapter is not powered	Power the adapter or replug to the power supply.
	Charging case is connect to the USB cable incorrectly.	Check and connect charging case to USB cable correctly.
	Battery over-discharging.	Charge the charging case for a period of time.
Charging case can not charge the hearing amplifiers.	Hearing amplifiers are not inserted into the correct slot of charging case.	Insert hearing amplifiers into the correct slot of charging case to charge.
	Charging contacts are dirty	Clean the charging contacts of hearing amplifiers and charging case.

7. [TECHNICAL SPECIFICATIONS]

1) Hearing amplifiers technical specifications	
Test Item	Value
Max. Output(OSPL90)	≤117dB
Equivalent input noise level (EIN)	≤32dB
The total harmonic distortion value(THD)	≤5%
Frequency response bandwidth	250Hz~5000Hz
Operation environment	Temperature: 0°C~ 40°C, Relative humidity:30%-75%RH, Atmospheric pressure range: 86~106kPa
Storage & transportation environment	Temperature: -10°C~+ 55°C, Relative humidity 10%-90%, Atmospheric pressure range: 70-106kPa
2) Battery life	
Test Items	Value

Bluetooth disconnected	≥12h
------------------------	------

- The actual battery life depends on battery quality, use pattern, active feature set, hearing loss and sound environment.
- The data are derived from the measured value of the product at the laboratory, not guaranteed values.

X. CAUTION

CE Requirements

(Simple EU declaration of conformity)

- HENAN ESHOW ELECTRONIC COMMERCE CO., LTD declares that the radio equipment type is in compliance with the essential requirements and other relevant provisions of RED Directive 2014/53/EU and the ROHS Directive 2011/65/EU and the WEEE Directive 2012/19/EU; the full text of the EU declaration of conformity is available at the following internet address: www.retekess.com.

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